



Report for CoViDiag+ , Infected patient#1 (A5)

Sample: Infected patient#1
Lot number:
Well number: A5
Configuration: CoViDiag + 2.0 (2.0)

Image before analysis:

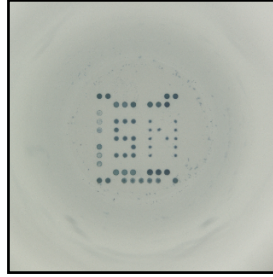

Analyzed image:

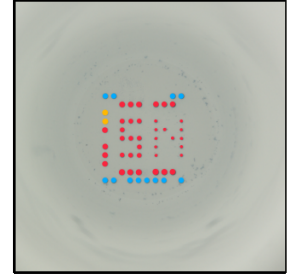

Replicate name	Result	Value
SARS-CoV-2		Intensity [AU]
Nucleocapside	Positive	55.276
Spike 1	Positive	67.049
Spike 2	Positive	77.952
Spike NTD	Positive	73.481
RBD UK	Positive	63.476
RBD SA	Positive	21.884
Spike RBD wt	Positive	54.103
RBD BRA	Positive	38.350
		Intensity [AU]
Control +	Positive	73.573 VALID (min. 10.000 AU)
SampCtrl	Positive	52.527 VALID (min. 10.000 AU)

Test name	Result	Comment
SARS-CoV-2	Positive	

Test is valid.




Report for CoViDiag+ , Vaccinated patient (F5)


Sample: Vaccinated patient
 Lot number:
 Well number: F5 †
 Configuration: CoViDiag + 2.0 (2.0)

† grid changed by user

Image before analysis:

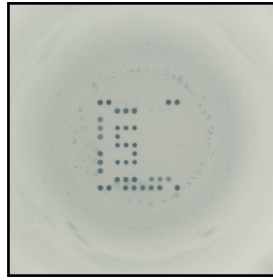

Analyzed image:

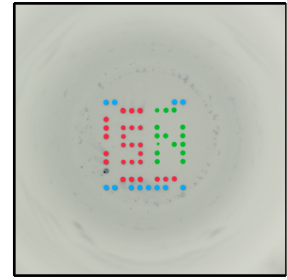

Replicate name	Result	Value
SARS-CoV-2		Intensity [AU]
Nucleocapside	Negative	0.855
Spike 1	Positive	67.157
Spike 2	Positive	40.590
Spike NTD	Negative	2.929
RBD UK	Positive	63.858
RBD SA	Positive	48.199
Spike RBD wt	Positive	62.681
RBD BRA	Positive	60.522
		Intensity [AU]
Control +	Positive	70.266 VALID (min. 10.000 AU)
SampCtrl	Positive	54.432 VALID (min. 10.000 AU)

Test name	Result	Comment
SARS-CoV-2	Positive	


Test is valid.




Report for CoViDiag+ , Infected patient#2 (C6)

Sample: Infected patient#2
 Lot number:
 Well number: C6
 Configuration: CoViDiag + 2.0 (2.0)

Image before analysis:

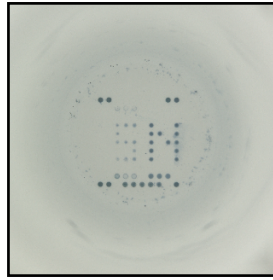

Analyzed image:

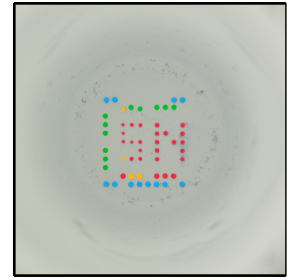

Replicate name	Result	Value
SARS-CoV-2		Intensity [AU]
Nucleocapside	Positive	56.851
Spike 1	Positive	26.092
Spike 2	Positive	52.092
Spike NTD	Negative	0.000
RBD UK	Negative	8.617
RBD SA	Negative	0.685
Spike RBD wt	Positive	22.000
RBD BRA	Negative	0.000
		Intensity [AU]
Control +	Positive	72.450 VALID (min. 10.000 AU)
SampCtrl	Positive	57.474 VALID (min. 10.000 AU)

Test name	Result	Comment
SARS-CoV-2	Positive	

Test is valid.



Report for CoViDiag+ , Non immunized patient (H6)

Sample: Non immunized patient


Lot number:

Well number: H6 †


Configuration: CoViDiag + 2.0 (2.0)

† grid changed by user

Image before analysis:

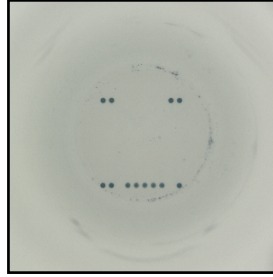

Analyzed image:

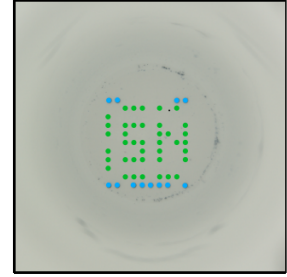

Replicate name	Result	Value
SARS-CoV-2		Intensity [AU]
Nucleocapside	Negative	0.081
Spike 1	Negative	0.146
Spike 2	Negative	0.000
Spike NTD	Negative	0.053
RBD UK	Negative	0.000
RBD SA	Negative	0.078
Spike RBD wt	Negative	0.090
RBD BRA	Negative	0.000
		Intensity [AU]
Control +	Positive	76.609 VALID (min. 10.000 AU)
SampCtrl	Positive	61.480 VALID (min. 10.000 AU)

Test name	Result	Comment
SARS-CoV-2	Negative	

Test is valid.

